R8661 - R8662
R8663 - R8671
R8672 - R8673
R8682 - R8683
R8684 - R8685

Changing Table – Operation and maintenance manual	en
Puslebord – Brugs- og vedligeholdelsesvejledning	dk
Wickeltische – Gebrauchs- und Pflegeanleitung	de
Tables à langer – Manuel d'utilisation et d'entretien	fr
Aankleedtafels – Gebruiks- en onderhoudshandleiding	nl
Skötbord – Bruks- och underhållsanvisning	sv
Stellebord – Bruks- og vedlikeholdsveiledning	no
Cambiador - Instrucciones de uso y mantenimiento	es
换洗床 - 操作和维护手册	中文

English	3
Dansk	22
Deutsch	40
Français	58
Nederlands	76
Svenska	94
Norsk	112
Español	130
中文	148

- en** The guarantee covers faults or defects in material or manufacture within a period of 3 years. Products are subject to minor technical modifications and design deviations. E. & O.E.
- dk** Garantien dækker i 3 år fra fakturadato for materiale- og fabriktionsfejl. Der tages forbehold for trykfejl, tekniske ændringer og modelafvigelser.
- de** Die Garantie gilt für drei Jahre ab Rechnungsdatum für Material- und Herstellungsfehler. Druckfehler, technische Änderungen und Modellabweichungen vorbehalten.
- fr** La garantie couvre les vices de fabrication et de matériaux pour une durée de 3 ans à partir de la date de facturation. Sous réserve de modifications techniques et d'erreurs d'impression.
- nl** De garantie is tot 3 jaar na factuurdatum geldig voor materiaal- en fabricagefouten. Onder voorbehoud van drukfouten, technische wijzigingen en modelafwijkingen.
- sv** Garantin gäller i 3 år från fakturadatum för material- och fabriktionsfel. Med reservation för tryckfel, tekniska ändringar och modellavvikelser.
- no** Garantien gjelder i 3 år fra fakturadato for material- og fabriktionsfeil. Med forbehold for trykkfeil, tekniske endringer og modellforskjeller.
- es** La garantía cubre fallos de material o fabricación durante los tres años posteriores a partir de la fecha de factura. Mantenemos reservas en caso de problemas de presión, modificaciones técnicas y variaciones de modelo.
- 中文** 保修涵盖 3 年内出现的材料或制造故障或缺陷。产品的技术和设计可能稍微发生变化，恕不另行通知。

Content

Symbols used in this manual.....	4
Symbol used on the product.....	4
General safety.....	5
Product labels.....	6
Intended use.....	8
Intended operator profile.....	8
Operating environment.....	8
Assembly and installation.....	9
Assembly and installation - accessories.....	10
Versions.....	11
Operating instructions.....	12
Safety tray.....	12
Height adjustment.....	12
Cleaning.....	13
Maintenance.....	14
Service inspection.....	14
Trouble-shooting.....	16
Resetting the changing table.....	16
Technical data.....	17
Dimensions (mm).....	18
Dimensions (inches).....	19
Accessories for changing tables.....	20
Disposal and recycling.....	21

en

IMPORTANT

RETAIN FOR FUTURE REFERENCE. READ CAREFULLY

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning.

Electricity / voltage warning.

Strong heat warning.

Symbol used on the product

Do not leave child unattended.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

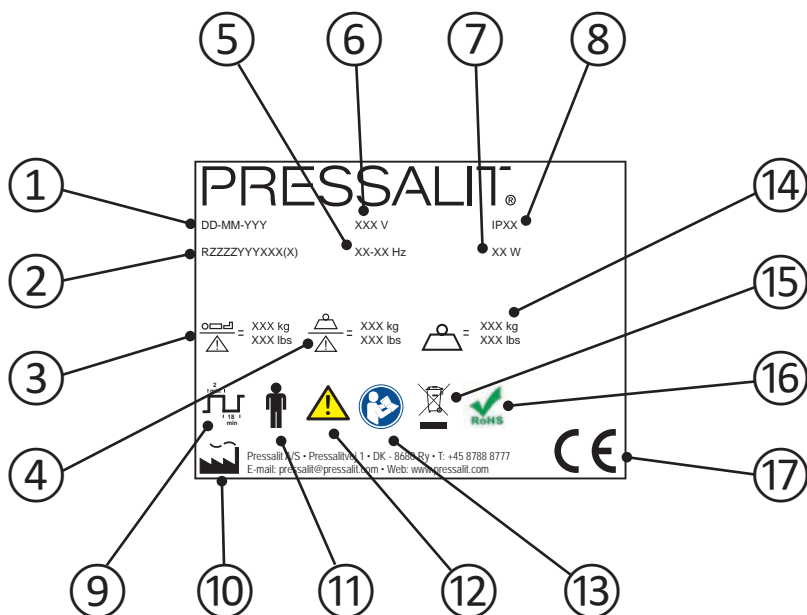
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

en

① Production date

② Article number

③ Maximum user weight

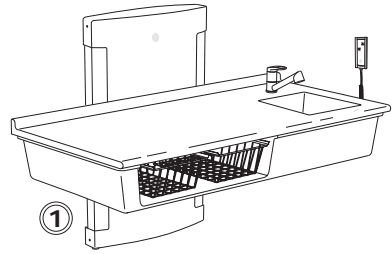
④ Safe working load

⑤ Frequency

⑥ Mains voltage

-
- ⑦ Maximum power consumption
-
- ⑧ IP class
-
- ⑨ Duty cycle The motor on height-adjustable products can be used for a maximum of 2 minutes of continuous operation, followed by 18 minutes pause.
-
- ⑩ Name and address of manufacturer
-
- ⑪ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
-
- ⑫ Consult the manual for use for important cautionary information such as warnings and precautions.
-
- ⑬ Consult manual for use
-
- ⑭ Product weight
-
- ⑮ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑯ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ⑰ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and with Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at www.pressalitcare.com.

Product label placement ①.

Intended use

This product is intended for optimum ergonomics when caring for children.
The child's weight must not exceed 75 kg / 165 lbs.

Intended operator profile

The product shall only be operated by adults.

Operating environment

The product is intended to be used indoor at room temperature under normal working conditions. For institutional use, the expected life time is 8 years.
Life cycle test performed according to ISO 17966:2016.

WARNING

Do not leave the child unattended.

Be aware of the risk of open fires and other sources of strong heat, such as electric bar fires, gas fires, etc. in the near vicinity of the changing table.

CAUTION

Never allow children to operate this product.

NOTICE

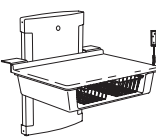
Always place the child on a changing table mattress and not directly on the table.

Assembly and installation

For assembly and installation, please refer to the assembly instruction supplied with the product.

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
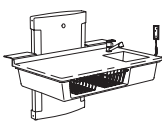
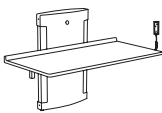
en

Assembly and installation - accessories

For assembly and installation, please refer to the assembly instruction supplied with the product.

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
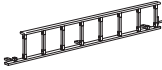
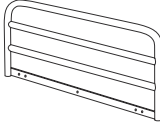
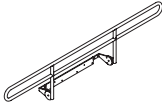
Versions

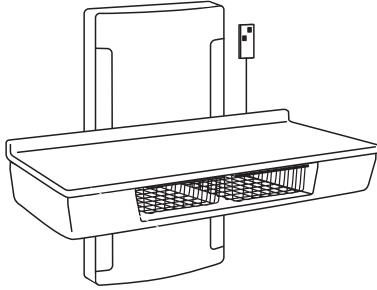
The changing table comes in various versions.

R8661, R8662, R8663

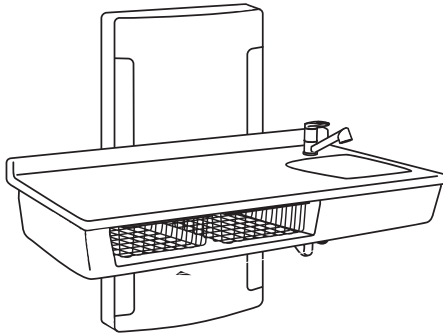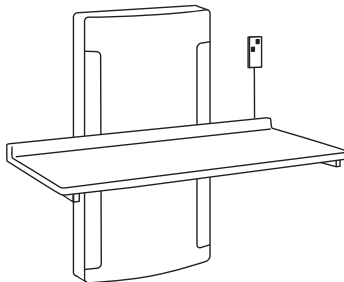With sanitary appliances: R8682, R8683, R8684, R8685

Folding version: R8671, R8672, R8673

Operating instructions

The changing table can be adjusted from 300 to 1000 mm / 11.81 to 39.37 inches up and down. **Max load is 75 kg / 165 lbs.**

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

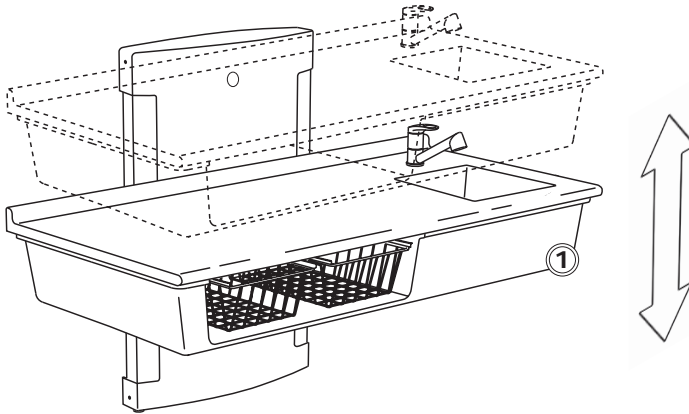
CAUTION

Never place objects underneath the changing table.

Safety tray

If the safety tray ① comes into contact with an object when the table is being lowered, the motor stops immediately and the table will rise 3-4 cm to reduce the risk of pinching/compression injuries under the table. The table can be used again as soon as the obstruction has been removed.

en

Height adjustment

The height adjustment is controlled by the wired hand control unit.

Press and hold the Up button to raise the changing table lifter, and press and hold the Down button to lower the table. When the button is released, the motion stops.

Cleaning

Clean the product after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chlorine-based cleaning or disinfecting products.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance

Expected service life of the changing table is 8 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

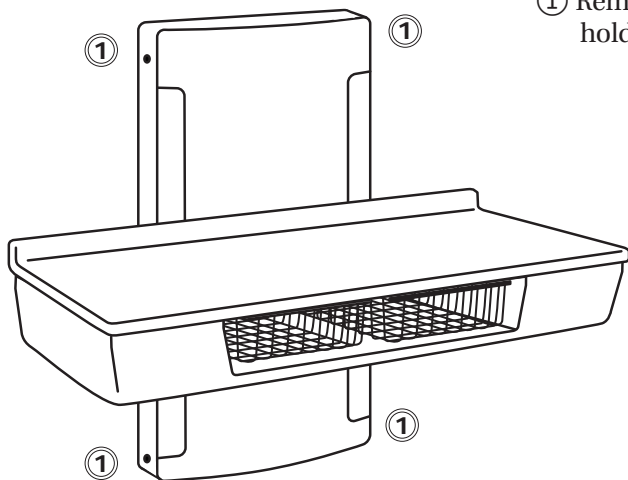
It is recommended that service inspection is done once a year.

Service inspection

Take the changing table to its highest position.


Disconnect mains before removing the covers.



① Remove the four screws ① holding the front cover.

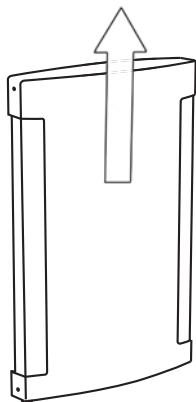en

② Lift off the top cover.

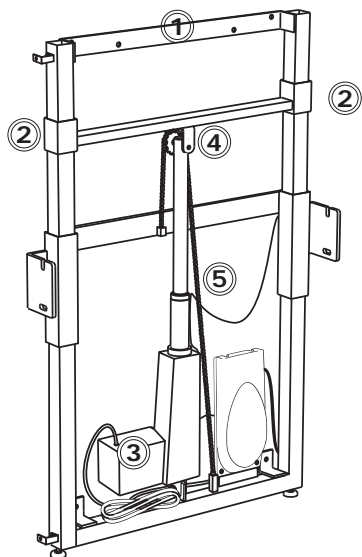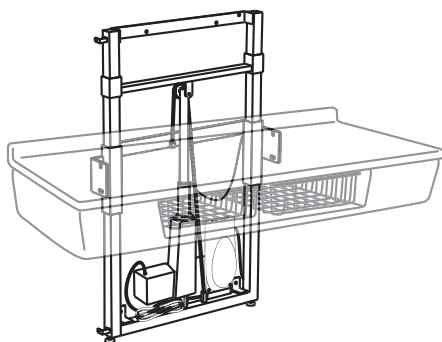③ Check the wall brackets ① and tighten if necessary.

④ Oil moving parts with WD40 or similar, especially the sliding brackets ②.

⑤ Check all electrical connections to the control box ③.

⑥ Check the chain wheel ④ for wear.

⑦ Oil the chain ⑤.

⑧ Put the unit cover back again and tighten the retaining screws. Reconnect mains.

Trouble-shooting

Symptom	Cause	Action
The product is jammed	Interruption of the power supply.	Reset the product. See below.
The product will not work, even though the control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Resetting the changing table


If the changing table is jammed, try to reset it.

- ① Make sure that the safety tray is not activated. Lower the changing table all the way down to its starting position.
- ② Release the lowering button, and after 3-4 sec. press it in again until you hear a slight click after approx. 10 sec.
- ③ Release the lowering button and then raise the changing table to its highest position. You will hear another click.
- ④ Lower the changing table a couple of centimetres.
- ⑤ Then raise it again to its highest position. You will hear another click. The changing table is now reset. The changing table should also be reset if there has been an interruption of the power supply.


Technical data

Max load	75 kg
Materials	Powder-laquered stainless steel Plastics
Weight	R8662: 65 kg / 143 lbs
	R8663: 75 kg / 165 lbs
	R8671: 55 kg / 121 lbs
	R8672: 60 kg / 132 lbs
	R8673, R8682, R8684:: 70 kg / 154 lbs
	R8683, R8685: 80 kg / 176 lbs
IP class	IP00
Power specifications	
Max consumption	75 W
Voltage	Mains connection: 230V
Frequency	50/60 Hz
Current	1.6 A

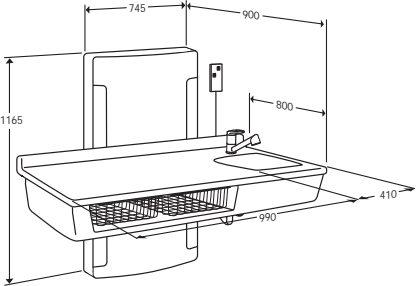
Dimensions (mm)


R8682, R8684

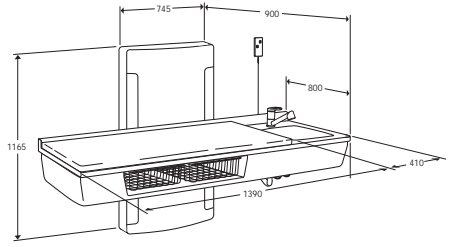R8683, R8685

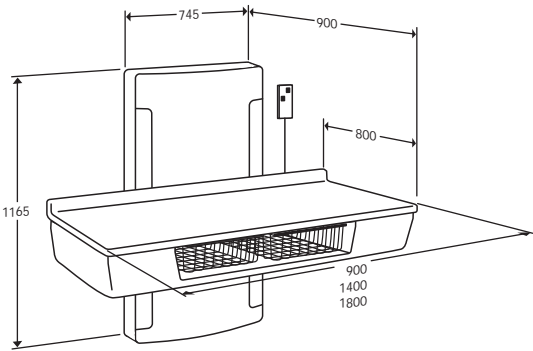
R8661, R8662, R8663

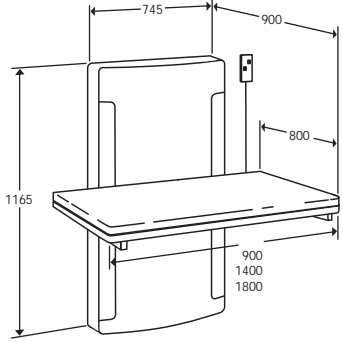
R8671, R8672, R8673

en

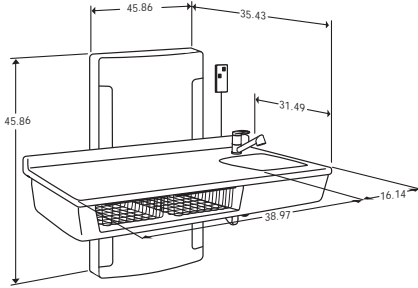
Dimensions (inches)


R8682, R8684

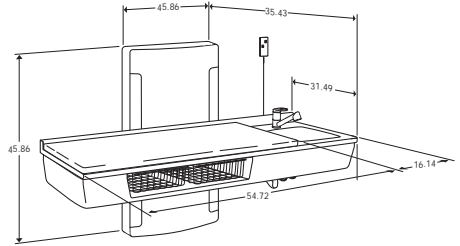R8683, R8685

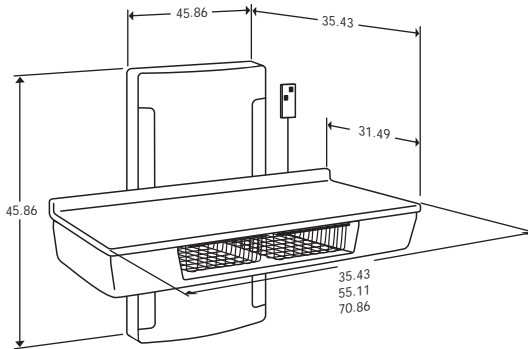
R8661, R8662, R8663

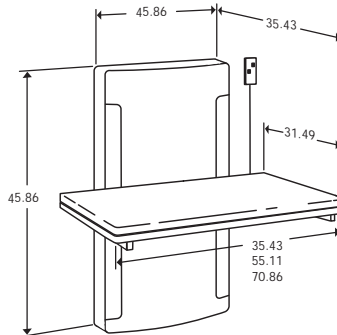R8671, R8672, R8673


en

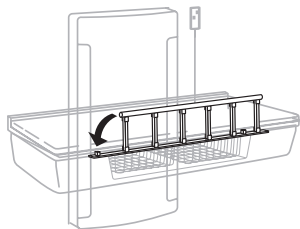Accessories for changing tables


RB1921 Guardrail

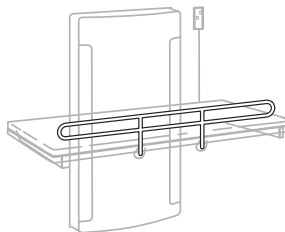Fits non foldable changing tables.
Length: 900 - 1400 mm / 35.43 - 55.11 inches


RB1922 Guardrail

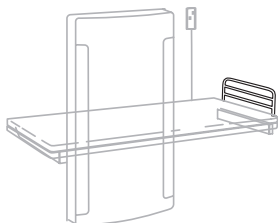Fits foldable changing tables.
Length is adapted after the length of the changing table.

R8707 End rail

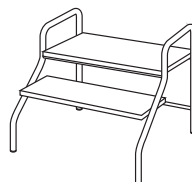
For mounting on changing tables, cannot be used on folding versions.
Length: 200 - 700 mm / 7.87 - 27.55 inches.

R8709 Stepladder

2 steps, width: 505 mm / 19.88 inches.

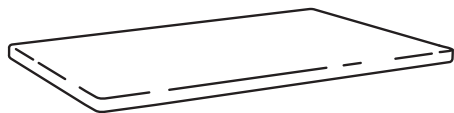en

Mattress for changing table

Available in 5 different lengths to fit the various changing tables.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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
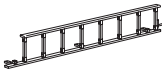
Subject to alteration without further notice. Not to be changed. For alterations and corrections, please contact us. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use.

keep living

Pressalit Care Telephone: +45 87 88 87 88
 Pressalit Care Fax: +45 87 88 87 88
 Pressalit Care E-mail: pressalit@pressalit.com
 Pressalit Care Web: www.pressalit.com

RB1707

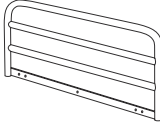PRESSALIT[®] care

(EN) Mounting instruction
 (EN) Monteringsvejledning
 (DE) Montageanleitung
 (FR) Notice de montage
 (NL) Montagevoorschrift
 (SE) Monteringsanvisning
 (NO) Monteringsveiledning
 (ES) Instrucciones de montaje


Subject to alteration without further notice. Not to be changed. For alterations and corrections, please contact us. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use.

keep living

Pressalit Care Telephone: +45 87 88 87 88
 Pressalit Care Fax: +45 87 88 87 88
 Pressalit Care E-mail: pressalit@pressalit.com
 Pressalit Care Web: www.pressalit.com

RB1922

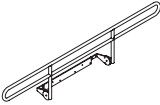PRESSALIT[®] care

(EN) Mounting instruction
 (EN) Monteringsvejledning
 (DE) Montageanleitung
 (FR) Notice de montage
 (NL) Montagevoorschrift
 (SE) Monteringsanvisning
 (NO) Monteringsveiledning
 (ES) Instrucciones de montaje

Subject to alteration without further notice. Not to be changed. For alterations and corrections, please contact us. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use. Please refer to the instructions for use.

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Rechte der Accession vorbehalten. © Hersteller der hier abgebildeten Waren. Änderungen vorbehalten.